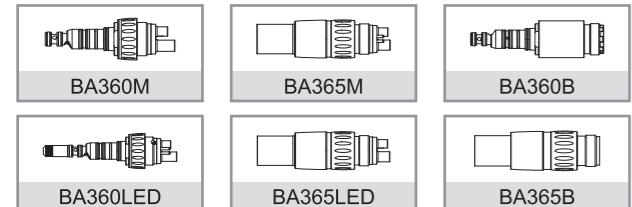
BA360 Optima-koppelstukken

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BA360B - BA365B

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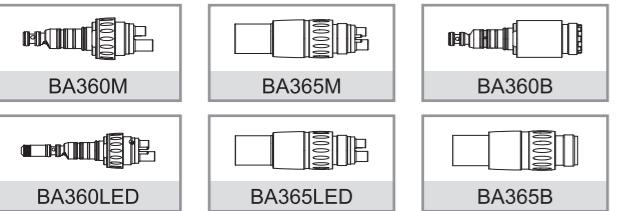
1. Product Description

BA360B – BA365B

Standard 2-hole connection, anti-suction valve.

BA360M – BA365M

Standard 4-hole connection, anti-suction valve.

BA360LED/BA365LED

Standard 4-hole connection, anti-suction valve, two electrical contacts and LED Bulb.

LED Bulb ideal voltage range: 3.0–3.4 Vd.c. (BA360LED)

LED Bulb recommended voltage: 3.2 Vd.c. (BA360LED)

LED Bulb ideal voltage range: 3.0–3.4 Vd.c. (BA365LED)

LED Bulb recommended voltage: 3.2 Vd.c. (BA365LED)

Table of BA360 couplings and suitable handpieces

Handpiece	Couplings
KAVO® LUX handpiece	BA360LED
KAVO® handpiece	BA360M/B
NSK Mach® LUX handpiece	BA365LED
NSK Mach® handpiece	BA365M/B

2. Safety Notes

- The BA360 couplings are intended only for use within dentistry as part of dental treatment; any form of misuse or modifications to the product are prohibited and may cause danger.
- Once BA360 couplings are damaged (ie deformed cause by hit) or improperly used (ie sterilized above suggested temperature), stop using them.
- Before operation, always make sure the product and its accessories meet the specified requirements.
- It is the duty of the user to use only faultless tools, for the correct purpose, and protect patients and third parties from all dangers.
- Please do not look directly into the lighting bulb, or it may hurt eyes when you use the BA360 coupling with LED.

3. Installation

Connection to the hose

- Attach the BA360 coupling to the hose.
- Use fixed tool to fix the BA360 coupling.
- Rotate the thread ferrule of the hose in a clockwise direction.
- Spray oil onto the O-rings. (for BA360M/B/LED)

Removal from the hose

- Use fixed tool to remove the BA360 coupling.
- Rotate the thread ferrule of the hose in an anti-clockwise direction.
- Separate the BA360 coupling from the hose.

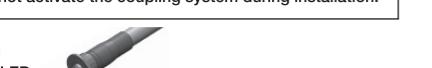
Connect the handpiece

Connect the handpiece exactly onto the BA360 coupling until it clicks audibly.

Removal from the handpiece (for BA365M/B/LED)

Pull back the BA360 coupling ring and remove from the handpiece.

Coupling ring

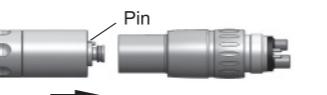
Do not activate the coupling system during installation.

Fixed Tool

BA360M/B/LED

2. BA365M/B/LED
(1) Connection to the coupling
Connect the fixed tool to the coupling. Ensure that the pin of the fixed tool is firmly inserted to the coupling hole until you hear the "Click" sound.

- (2) Approach in the use of the coupling
> as a coupling wrench

> as a protection-against-dust tool

> as a bulb replacement wrench

4. Trouble Shooting

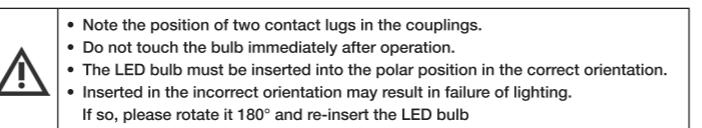
Replacing the bulb (for BA360LED)

Unscrew the cap by turning it anti-clockwise.
Pull a bulb out of the coupling and insert a new one.
Screw the cap by turning it clockwise.

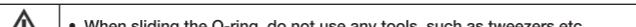
Replacing the bulb (for BA365LED)

Refer to Fixed Tool section.

Replacing the O-ring (for BA360M/B/LED)

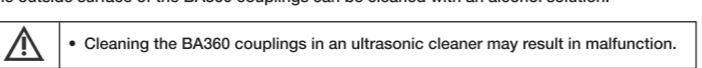
Press the sides of a defective O-ring between two fingers until a loop is formed and then slide the O-ring out of the couplings and remove it. Insert these new O-rings in the grooves and spray them with lubrication oil.

5. Cleaning, Disinfection and Sterilization

Cleaning exterior

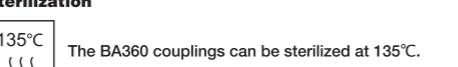
The outside surface of the BA360 couplings can be cleaned with an alcohol solution.

Disinfection

The outside surface of the BA360 couplings can be disinfected with chemical disinfectants. Do not immerse BA360 couplings in disinfectants.

Sterilization

6. Disclaimer

- Information in this operation manual was carefully reviewed. We cannot assume any liability for damage arising from improper use of the handpiece or from use of this manual.
 - Products are subject to change without notice.
 - This product is only for dental handpiece treatment and use for other purpose may damage the handpiece and cause severe consequence.
 - Please use the product correctly according to the instruction.
- KAVO® & MULTIflex® are the registered trademarks of Kaltenbach & Voigt GmbH CO, Germany
NSK Mach® are the registered trademark of NAKANISHI INC. Japan

7. Quality and Service

All of our products have passed strict QC procedures before delivery. Please note that improper use or not using the recommended parts and accessories or modifying our products without our approval may shorten the lifetime of the product or cause damage. Please contact us or dealers with proof of purchase for consultation whenever you have any questions.

CE

MANUFACTURED BY/FABRICADO POR:

B.A. INTERNATIONAL LTD.

Unit 9, Kingsthorpe Business Centre

Studland road, Northampton - NN2 6NE U.K.

0JL0085BA1_V4