# SuPreme S300

## *Light-Curing-Unit*

MODEL: BA190020

**Operating instructions**  
**Modo de empleo**  
**Mode d'emploi**  
**Betriebsanleitung**  
**Gebbruikershandleiding**  
**Istruzioni per l'uso**

 **B.A. INTERNATIONAL LTD.**  
(manufacturer)  
Unit 9, Kingsthorpe Business Centre,  
Studland Road, Kingsthorpe,  
Northampton NN2 6NE U.K.

Rev.2015/09

# Operating instructions

## SuPreme S300

### Light-Curing Unit

- 1 Connection cable (back of the unit)
- 2 ON/OFF switch
- 3 Polymerization chamber
- 4 Drawer
- 5 Option key curing time
- 6 Display curing time
- 7 Nipple

## Operating instructions

The light-curing device was developed for use in the polymerization of light-curable dental materials like universal trays, sealing lacquer and block-out materials. Homogeneous illumination by 4 UVA fluorescent tubes incorporated into a high-performance aluminium reflector. Three curing times can be selected: 5 minutes, 10 minutes or continuous operation.

## 1. GENERAL

1.1 These instructions facilitate the safe and efficient handling of the device. The instructions form a component part of the light-curing unit and should be kept in an appropriate place accessible to the personnel in the vicinity of the unit. Before starting all tasks, personnel must carefully read and understand these instructions. The basic prerequisite for safe working is compliance with all the safety and handling instructions described herein. Furthermore, the local accident prevention regulations and general safety conditions for the application of the device are also applicable.

### 1.2 Limitations of liability

The manufacturer assumes no liability for damage due to

- non-observance of the operating instructions,
- unintended use,
- employment of untrained personnel,
- technical modifications,
- use of unapproved spare parts. We reserve the right to make technical changes in the framework of improvement and further development.

### 1.3 Copyright protection

These operating instructions are copyright protected. Passing these instructions on to third parties, reproduction of any kind – even in form of excerpts – as well as the use and/or disclosure of the contents without the written consent of the manufacturer is not permitted, except for internal purposes.

### 1.4 Spare parts

Incorrect or faulty parts can compromise safety and cause damage.

- Only use original spare parts from the manufacturer.

### 1.5 Guarantee provisions

The relevant statutory regulations apply.

### 1.6 After-sales service

Our customer service department is available to provide technical information. Furthermore, our employees are always interested in new information and experience arising from use and which can be valuable for the improvement of our products.

## 2. SAFETY

### 2.1 Responsibility of the operator

The light-curing unit is exclusively used for commercial purposes. Its operator must therefore comply with all applicable statutory obligations regarding occupational health and safety. The operator has to ensure that all personnel working on or with the device have read and understood these instructions. Only appropriately trained personnel are allowed to use the device.

### 2.2 Intended use

The light-curing device is exclusively conceived and designed for curing dental materials: it can be chosen between a curing time of 5 minutes, 10 minutes and continuous operation.

### 2.3 Safety instructions

- When installing and employing UVA/UV lights, always observe the local authority's recommendations for use and the health and safety regulations applying locally.
- The material to be polymerized should always be inserted when the fluorescent lights are switched off.
- After a service life of approximately 1000 hours, the radiant power of the lamps is reduced, thus extending the curing times. Please observe the duty cycle or exchange the tubes.

#### **Attention! Disconnect the power plug first.**

- Changing the UVA light tubes: Press the nipple (7) down and remove the drawer completely.
- Always install 4 UVA lamps.
- After installing the UVA lamps, place the drawer (4) exactly in the middle of the rail track and press it in.
- Disposal must be effected in accordance with the applicable legal regulations (waste electrical and electronic equipment).

### 3. ENTRY INSPECTION

The unit and its accessories must be immediately examined upon receipt for defects and shipping damage. Claims for damages can only be filed if the seller and the forwarding agent have been immediately notified.

### 4. TECHNICAL DATA

Power supply	230 V AC 50 Hz $\pm$ 10 %
Maximum power consumption	approx 36 W
Unit fuse	T 1.25 A
Spectral range	350-500 nm
Dimensions	245 x 250 x 170 mm (width x depth x height)
Number of lamps	2 UVA lamps 350-400 nm and 2 UVA lamps 400-500 nm

### 5. OPERATION

- 5.1 Connect the mains cable (1).
- 5.2 Place the model in the drawer (4).
- 5.3 Switch on the device using the mains switch (2) at the back.
- 5.4 Press the key (5) to select the curing time:
  - a) 5 minutes,
  - b) 10 minutes,
  - c) continuous operation. The curing time selected last is automatically saved.
- 5.5 Insert the drawer (4) fully into the device; the UVA lamps are automatically switched on.

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